

Dental Model

Instructions for Use

This product should be used by the operator in accordance with the instructions of the user manual and within the scope of application. The operator should read this manual carefully before using it.

【Product Name】 Dental Model

【Structure and composition】 This product is a resin material.

【Intended purpose】 Used to make dental models

【Specifications】 1.0 kg/bottle, 2.5 kg/bottle, 5.0 kg/bottle, 10 kg/bottle

【Model】 M 2.0, M 3.0

【Operation description】

1. This product is a 3D printing material suitable for DLP printers. Please shake well before use.
2. Please wear goggles and rubber gloves when using.

【Storage conditions and methods】

The product should be kept away from light, moisture and sealed under the condition of 5-25°C.

【Precautions】

Those who are allergic to acrylic resin ingredients should not be allowed. If allergic reactions occur during use, please consult a doctor.

【Adverse events】 There are no clinical cases of adverse events for this product.

【Shelf life】 12 months

【Accessories】 No

【Symbols used in packaging/labeling】



—Temperature limit;



—Keep away from sunlight;



—Keep dry;



—Use-by date;



—Batch code;



—Caution;



—Medical device



—Authorized representative in the European Community



Shandong Huge Dental Materials Corporation.

No. 68 Shanhai Road, Donggang District, Rizhao City, Shandong Province, 276800, P.R. China

Tel: +86 633 2277268

Fax: +86 633 2277298

www.hugedental.com

marketing@hugedental.com



MedNet EC-REP GmbH

Borkstrasse 10, 48163 Muenster, Germany